

KDB **GLIDE**®


*smooth* precision in your hands



## FEATURES

KDB Glide's unique design and dimensions enable precise excision of diseased trabecular meshwork (TM), allowing for increased access to collector channels

RAMP & DUAL BLADES

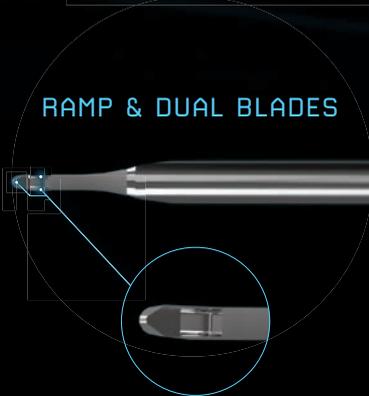

ROUNDED HEEL



TAPERED SIDES & FOOTPLATE



KDB Glide's proprietary **features** are designed to provide optimal interface with the canal of Schlemm for excisional goniotomy

**Ramp & Dual Blades:** Ramp facilitates lift and stretch of the TM, while dual blades create parallel incisions for clean TM excision.


**Rounded Heel:** Smooth transition through the canal of Schlemm when performing excisional goniotomy.

**Tapered Sides & Footplate:** Allows for optimal interface with the canal of Schlemm, designed to treat more patients with variable anatomy.

# smooth precision


KDB Glide enables you to perform excisional goniotomy using a proven implant-free technique that only requires 90-120 degrees of treatment

## MARK and MEET TM Excision Technique

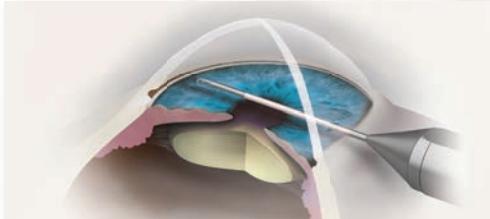



### STEP 1

Introduce the KDB Glide through a clear corneal incision.

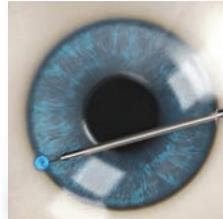


### STEP 2

Under gonioscopic visualization, engage the TM with the pointed tip at a 10° angle to canal of Schlemm to **MARK** the excision end point.

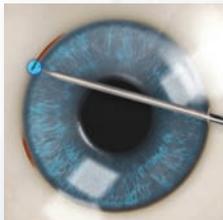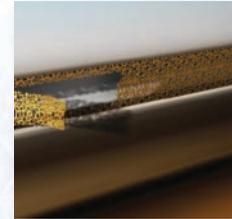

### STEP 3

Disengage the KDB Glide, then re-engage it 3 to 4 clock hours from the initial TM incision, once again with the pointed tip at a 10° angle to canal of Schlemm.

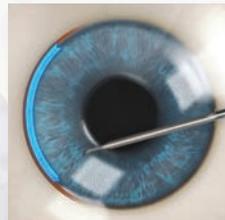


### STEP 4

Once the KDB Glide is re-engaged, seat the foot plate, then advance the dual blades through the planned excision to **MEET** the initial **MARK** point.



### STEP 5

Remove the detached excised TM tissue.

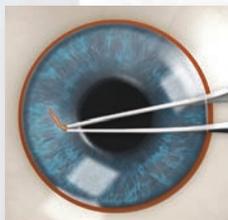

### Scan & Watch

Scan the QR code to watch the procedure.



### Ready to get started?

Join **LEARN**, our online training platform, to instantly gain access to product education and training modules at [www.learn.newworldmedical.com](http://www.learn.newworldmedical.com)

For more information, contact your local sales representative or email us at [customercare@labtician.com](mailto:customercare@labtician.com)

